


BetaSpecs™

INSTRUCTIONS FOR USE

... Disposable frameless eyeshield

CAUTION: BETASPECS™ ARE NOT DESIGNED FOR SAFETY AND MUST NOT BE USED IN PLACE OF SAFETY GLASSES

Figure 1

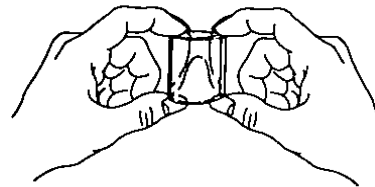

1. Remove BetaSpecs™ from package and hold as shown in Figure 1, with nose-notch facing away from you.

2. Holding the eyeshield by its edges, pull the ends apart as shown in Figure 2. To avoid finger smudges, always handle BetaSpecs™ by the edges.

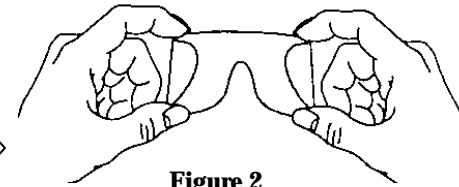


Figure 2



Figure 3

3. Place BetaSpecs™ over the eyes as shown in Figure 3. The eyeshield is held in place with the gentle curling pressure.

4. BetaSpecs™ may be worn with spectacles. Simply remove spectacles, position eyeshield as shown above, and replace spectacle.

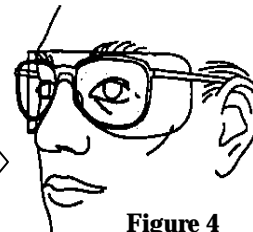


Figure 4

For more information
by phone: +1 847 267 5929 (USA)
+44 (0) 20 8903 5127 (UK)
by email: info@gammahealth.com
www.gammahealth.com

